

Inspection checklist

Refer to the Operating Instructions for the User's Checklist

- Inspect the defibrillator for signs of damage, cracks or foreign substances
- It is not necessary to switch on the defibrillator for the inspection
- Observe the Readiness Display on the top right and perform the following checks:
 1. The OK symbol is visible
 2. The battery symbol shows two or more bars
- Confirm spare, fully charged battery is readily available and in-date
- Ensure there is one pack of QUIK-COMBO pads preconnected to the defibrillator, and the pack of pads is sealed and in-date
- There should be one additional set of adult pads
- If present, check Infant/Child Reduced Energy Defibrillation Electrodes to make sure they are sealed and in-date

Readiness display

OK indicates device is ready for use.



Battery is fully charged.



Battery is very low. Replace immediately.



Battery is depleted. Replace immediately.



Service symbol indicates service is required.

stryker

LIFEPAK[®] 1000 Defibrillator

Pocket guide



This is not the complete instructions for use. Refer to Operating Instructions for complete device information including directions for use, intended use and warnings.



The LIFEPAK 1000 defibrillator performs an automatic self-test every 24 hours. A Service Alert tone will sound and the Service symbol will be displayed if a fault is detected.

For a brief summary of indications and important safety information, visit our website at www.strykeremergencycare.com/disclosure-and-safety-information/lifepak-1000-defibrillator/

For further information, please contact your local Stryker representative or visit our website at www.strykeremergencycare.com

 **Physio-Control, Inc.**, 11811 Willows Road NE, Redmond, WA 98052 U.S.A.

Using the LIFEPAK® 1000 as an AED

Follow these 3 steps:

1. Apply QUIK-COMBO® Redi-Pak™ electrodes to patient's chest
2. Press **On** button, Auto-analyze will take place
3. If a shock is advised, press **Shock** button



Adult QUIK-COMBO electrodes with REDI-PAK Preconnect System



Infant/Child Reduced Energy Defibrillation Electrodes with the pink cable connector deliver attenuated energy to children less than 8 years of age or less than 25 kg (55 lbs)